Kundenservice DE

Web: www.dspro.de/kundenservice

+49 (0) 38851 314650*

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List of Contents

Items Supplied _____	16
The Device at a Glance _____	16
Information about your Operating Instructions _____	17
Intended Use _____	18
Safety Notices _____	18
Description of the Device _____	20
Operating Elements and Functional Overview _____	21
Filling the Tank _____	22
Setting Up and Connecting the Device _____	22
Use _____	23
Replacing the Battery _____	24
Cleaning and Storage _____	25
Troubleshooting _____	26
Technical Data _____	27
Disposal _____	27

IMPORTANT NOTICES!
PLEASE KEEP FOR REFERENCE!

Items Supplied

- Main unit (1 x)
- Tank (1 x)
- Filter holder with inserted filter sheet (1 x)
- Spare filter sheet (1 x)
- 3 V battery CR2032 (1 x)
- Operating instructions (1 x)

Check the items supplied for completeness and the components for transport damage. If you find any damage, do not use the device but contact our customer service department.

The 3 V battery CR2032 is already inserted into the device on delivery, and provided with a protective strip. Remove before initial use

The Device at a Glance

Main unit

(Fig. A)

- | | |
|--|---|
| 1 Button SET | 7 Battery compartment (on bottom) |
| 2 Display | 8 Tank |
| 3 Button  | 9 Safety catch |
| 4 Button  | 10 Ventilation slots |
| 5 Tank handle | 11 Button  |
| 6 Liquid level indicator | |

Display

(Fig. B)

- | | |
|---|---------------------------|
| 12 Hour display | 16 Fan speed - low |
| 13 Minute display | 17 Fan speed - high |
| 14 Marks the start time of the timer together with the time | 18 Fan speed - night mode |
| 15 Marks the end time of the timer together with the time | 19 Filling the tank |
| | 20 Fan defect |

Key numbers are displayed as follows: (1)

Image references are shown as follows: (Fig. A)

Congratulations!

Dear Customer,

We are pleased that you have chosen this **Thermal Brine Evaporator**.

If you have any questions about the device or spare parts/accessories, please contact our customer service department via our website: **www.dspro.de/kundenservice**

We hope you will enjoy using your new device!

Information about your Operating Instructions

Before using this device for the first time, read the operating instructions carefully and keep them for future reference. They form an integral part of the device. If the device is passed on to others, the operating instructions should be passed on together with the device.

The manufacturer and importer do not accept any liability if the information in these operating instructions is not complied with.

The illustrations in these operating instructions may differ from your device.

Explanation of the symbols

Danger symbols: These symbols indicate dangers of injury. Read through the associated safety notices carefully and follow them.

Symbol for "Type BF" classified equipment (degree of protection against electric shock)

Circuit symbol for alternating current

Temperature range

Read operating instructions

IP21

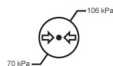
Protection type

Devices bearing this mark were placed on the market after 13.08.2005 (DIN EN 50419). Devices marked in this way meet the labelling requirements of all EU member states.

Protection class II device

Air pressure range

Humidity range

Supplementary information

Serial number

With the CE marking, the manufacturer or importer confirms that the device complies with the applicable EC directives and the "essential requirements" specified in them

Explanation of the signal words

Signal words indicate hazards if the associated notices are not followed.

DANGER – high risk, results in serious injury or death

WARNING – medium risk, may result in serious injury or death

NOTICE – may result in risk of damage to material

Intended Use

- The device is intended for the evaporation of thermal brine.
- **The device is intended exclusively for use with our DS VieGlobal thermal brine.** No other liquids should be poured into the device.
- The device should only be used as described in the operating instructions. Any other use is deemed to be improper.
- The device is for personal use only and is not intended for commercial applications.
- The warranty does not cover faults caused by incorrect handling or damage. The same applies to normal wear and tear.

Safety Notices

Warning: Please note all safety notices, instructions, illustrations and technical data provided with this device. Failure to follow the safety notices and instructions may result in electric shock, fire and / or injuries.

- This device may be used by **children** aged 8 and over and by people with reduced physical, sensory or mental abilities or a lack of experience and/or knowledge if they are supervised or have received instruction on how to use the device safely and have understood the dangers resulting from failure to comply with the relevant safety precautions.
- Children must **not** play with the device.
- Cleaning and **user maintenance** must not be carried out by **children**, unless they are older than 8 years of age and are supervised.
- Children younger than 8 years of age and animals must be supervised in the vicinity of the device.
- The device is not designed to be operated with an external timer or a separate remote control system.
- Do not make any modifications to the device. The housing of the device must not be opened. Only the water tank may be removed.

- If the device or the connecting cable are damaged, they must be replaced by the manufacturer, customer service department or a similarly qualified person (e.g. specialist workshop) in order to avoid any hazards.

DANGER – Danger of Electric Shock

- Use and store the device only in closed rooms.
- Do not use the device in rooms with high humidity.
- Never immerse the main unit and the connecting cable in water or other liquids, and ensure that these components cannot fall into water or become wet.
- Should the main unit fall into water, switch off the power supply immediately. Do not attempt to pull it out of the water while it is connected to the mains power! Have the device checked by a specialist workshop before putting it back into operation.
- Never touch the main unit and the connecting cable with wet hands when the device is connected to the mains power.
- Do not insert any metal objects (e.g. knives, tools or the like) into the openings of the device, as they can cause an electric shock / short circuit.
- The device must always be disconnected from the mains power when it is not being supervised, when filling or emptying the water tank, and before cleaning and changing the filter sheet.

WARNING – Danger of Injury

- Keep the packaging material away from children and animals. There is a danger of suffocation.
- Make sure that the connecting cable is always kept out of the reach of small children and animals. There is a danger of strangulation.
- Batteries can be deadly if swallowed. Batteries and the remote control should be kept out of the reach of children and animals. If a battery is swallowed, immediate medical attention must be sought. There is a danger of burns from battery acid.
- If a battery leaks, avoid physical contact with the battery acid, and wear protective gloves if necessary. If you touch battery acid, rinse the affected areas with abundant clear water straight away and seek medical assistance immediately. There is a danger of burns from battery acid.
- Make sure that the connecting cable does not present a trip hazard.
- Inspect the device for damage and/or wear and tear each time before you use it. It may only be used if it is undamaged and does not display any signs of wear and tear.

WARNING – Danger of Fire

- Do not use the device in rooms containing easily ignitable or explosive substances.
- Do not operate the device in the vicinity of combustible material. Do not place any combustible materials (e.g. cardboard, paper, plastic) on the device.
- Do not cover the device during operation in order to prevent it from catching fire. Do not insert anything into the ventilation openings of the device and make sure that they do not become clogged.

NOTICE – Risk of Damage to Material and Property

- Do not tilt the device if there is thermal brine in the tank.
- Switch off the device and pull the mains plug out of the socket if an error occurs during operation, when there is a power cut or before a thunderstorm.
- Do not use the device if it has malfunctioned, fallen or been dropped into water. Have it checked in a specialist workshop before using it again.
- The device must be switched off when inserting or removing the mains plug.
- Never pull or carry the device by the connecting cable. When you pull the mains plug out of the socket, always pull on the mains plug and never the mains cable!
- Protect the device from other heat sources, naked flames, sub-zero temperatures and impacts. Also protect the main unit from moisture and wet conditions.
- Use only original accessories from the manufacturer in order to guarantee that there is no interference that may prevent the device from working and to avoid possible damage.
- Do not place or lay any heavy objects on the device or the connecting cable.

Description of the Device

The evaporator is a device that essentially consists of a container for the liquid to be evaporated and an electronically controlled fan. The fan creates a draft which moves over the liquid to be evaporated and releases the evaporating substances into the surrounding ambient air.

The evaporator can be used up to 24 hours a day. The device is suitable for extended use.

During the evaporation of the thermal brine, crystal deposits can form on the tank (8). These are normal and do not represent a defect. Crystal formation also depends on the humidity and temperature of the room in which the device is used.

Operating Elements and Functional Overview

The device has the following operating elements and functions:

Button	Function	
Button  (11)	in the standard mode	
	Displaying timer settings	Activating the setting mode
	in the setting mode	
	Confirming settings and switching to the next display	
Button SET (1)	Increasing the level / displaying the options	
Button  (3)	in the standard mode	
	Setting the speed of the fan (when used without a timer)	
Button  (4)	Switching the device on/off	

- To activate the fast scan of the adjustable values in the setting mode, the **SET** button has to be kept pressed.
- By briefly pressing the  button the start and end time of the timer can be viewed.

Display illumination

The display illumination goes out approx. 30 seconds after the last button is pressed. Press any button to switch on the display illumination.

Fan

The fan has three different speed levels, which you can set in the standard mode by pressing the  (3) button several times. The levels can be set in the following order:

High speed  – Low speed  –
Night mode  – Off (no symbol)

The set level is indicated by the corresponding symbol in the display (2) (see “Device at a Glance” - “Display”).

When using the device with the timer, the speed level of the fan is set with the **SET** (1) button (see the “Use” - “Using the Device with the Timer” chapter).

Timer

If the device is used with a timer, the fan switches on at the set start time and switches off again automatically at the end time. For the device to work with the timer, the time has to be set and the device must be in standby mode.

If “12:00” appears in the display (2) after the device is switched on, the time has not yet been set. The device is in the factory setting.

Refill reminder

After 30 days the symbol lights up on the display (2)  to remind you to fill the tank or replace the liquid. In addition, a beep sounds every 5 minutes as a reminder. After filling, you have to press the buttons **SET** (1) and  (11) simultaneously for about 3 seconds to reset the 30-day countdown. The display shows "FULL".

Safety switch

The device has a safety switch which automatically places the device into standby if the device topples over. The device continues to emit warning tones until it is placed upright again.

Filling the Tank

PLEASE NOTE!

- Do not overfill the tank! The **MAX** mark on the tank must not be exceeded.
 - Thoroughly clean the tank before filling it again.
-
1. Make sure that the device is switched off and has been disconnected from the mains power!
 2. Push down the two safety catches (9) on the left and right of the housing. The tank (8) is detached from the main unit (**Fig. C**).
 3. Pull the tank out of the main unit.
 4. Fill the tank with the thermal brine.
 5. Insert the filled tank into the main unit.
 6. Press the two safety catches down and at the same time press the tank upwards in the main unit. In this position you release the safety catches to fix the tank in the main unit.
 7. After filling the tank, simultaneously press the buttons **SET** (1) and  (11) for approx. 3 seconds in order to reset the 30-day countdown. The display shows "FULL".

Setting Up and Connecting the Device

PLEASE NOTE!

- Only connect the device to a socket that is properly installed and matches the technical data on the type plate of the device. The socket must also be readily accessible after connection so that the connection to the mains can quickly be isolated.
 - Unwind the connecting cable fully before connecting up the device.
 - Lay the connecting cable in such a way that it does not become squashed or bent and does not come into contact with hot surfaces.
-
1. Place the device on a dry, even, solid and moisture-resistant surface with clearance of at least 30 cm above it and to all sides.
 2. Connect the device to a socket.

Use

PLEASE NOTE!

- **Familiarise yourself with the safety instructions of the thermal brine device before use!**
- The device must not be moved during use.

The device can be used either with or without a timer.

During the evaporation of the thermal brine no visible mist can be seen. This is normal and not a defect.

Using the Device without the Timer

1. Press the button (4) to switch the device on. The device is in standby mode.
2. Use the button (3) to set the desired speed level (see the “Operating Elements and Functional Overview” - “Fan” chapter).
3. To switch off the device, press and hold the button for about 3 seconds.

Using the Device with the Timer

To use the device with the timer, the following settings have to be made:

- Setting the time
- Setting the starting time of the timer
- Setting the end time of the timer
- Setting the speed of the fan

Please Note: The setting mode is automatically terminated if no button is pressed for approx. 30 seconds in the setting mode. The screen stops flashing and the display (2) changes to the standard display (current time). The settings made are saved. If any settings still need to be made, you have to reactivate the setting mode. Use the **SET** (1) button to jump directly to the desired setting display (Time > Start time > End time > Fan speed).

1. Press the (4) button to switch off the device. The device is in standby mode.
2. Press and hold the (11) button for approx. 3 seconds. You are in the time setting mode. The hour display flashes.

Setting the time (Fig. D1)

3. Press the **SET** (1) button until the desired hour is set.
4. Press the button to switch to minutes. The minute display flashes.
5. Press the **SET** button repeatedly until the desired minutes are set.
6. Briefly press the button to switch to the next display.
You are now in the setting mode for the timer start time. The hour display flashes and in addition the **ON** (14) symbol is illuminated.

Setting the start time (Fig. D2)

7. Press the **SET** button repeatedly until the desired hour is set.
8. Press the  button to switch to minutes. The minute display flashes.
9. Press the **SET** button repeatedly until the desired minutes are set.
10. Briefly press the  button to switch to the next display. The **ON** symbol goes off. The starting time is set.

You are now in the setting mode for the timer end time. The hour display flashes and in addition the **OFF (15)** symbol is illuminated.

Setting the end time (Fig. D3)

11. Press the **SET** button repeatedly until the desired hour is set.
12. Press the  button to switch to minutes. The minute display flashes.
13. Press the **SET** button repeatedly until the desired minutes are set.
14. Briefly press the  button to switch to the next display. The end time is now set.
You are now in the setting mode for the fan speed level. The display shows "FAN" and the symbol  (17) flashes.

Setting the fan speed (Fig. D4)

15. Press the **SET** button several times to switch between the different speed levels.
The levels can be set in the following order: *High speed*  – *Low speed*  – *Night mode* 
16. Briefly press the  button to exit the setting mode and switch to the standard display (current time). The timer is set and activated.

Please Note: To deactivate the timer, the starting and end time have to be identical. For this purpose you have to go through the individual steps of the timer setting again.

The settings remain stored until they are changed.

Replacing the Battery

PLEASE NOTE!

- Use only the battery type specified in the technical data. Otherwise there is a danger of explosion.
- The battery must not be charged or reactivated in any way, disassembled, thrown into a fire, immersed in liquids or short-circuited. Also protect the battery from heat and sunshine. There is a risk of explosion!
- The battery contains chemical substances. The battery should therefore always be disposed of properly.
- Take the battery out of the device if it is dead or if you will not be using the device for a prolonged period. This will prevent any damage caused by leaks.
- Do not use the device if the battery has leaked. Remove the leaking battery from the battery compartment immediately.

The device is powered by electricity. However, to be able to keep the settings when the device is disconnected from the power supply, a battery is also inserted.

Unless the device is connected to the power supply, the current time and timer settings will be deleted during battery replacement. We therefore recommend connecting the device to the mains before replacing the battery.

1. Remove the water tank (8) and empty it before replacing the battery
2. Using a suitable screwdriver, loosen the screw on the battery compartment (7) and remove the battery compartment cover.
3. Remove the used battery and insert a new battery (3 V; CR2032). The positive pole must be facing upwards.
4. Place the battery cover on the battery compartment and fix it with the screw.

If the settings are not saved after the device is disconnected from the power supply, the battery has to be replaced.

Cleaning and Storage

PLEASE NOTE!

- Clean the tank and the filter sheet every time you refill or change the thermal brine.
 - Do not use any caustic or abrasive cleaning agents to clean the device. These may damage the surfaces.
-
- Empty the tank and clean it before storing the device.
 - Clean the tank (8) with detergent and warm water.
 - Rinse the filter thoroughly under running water. Then allow it to air dry completely before putting it back in the main unit.
 - If necessary, dust the housing or wipe it with a damp cloth. Then dry with a cloth.
 - When not in use the device should be protected from dust in a clean, dry place out of the reach of children and animals.

Changing the Filter Sheet

The device must be kept operationally ready. Replace the filter sheet regularly (every 3 months) or when required.

New filter sheets can be purchased via the following website:

www.dspro.de/kundenservice

1. Remove the tank (8) from the main unit (see the “Filling the Tank” chapter).
2. Place the device in front of you.
3. The filter holder is located inside the main unit behind the fan (Fig. E1). Pull it out of the main unit by its handle (Fig. E2).
4. Open the back of the filter holder and remove the filter sheet (Fig. E3).
5. Place a new filter sheet in the filter holder and close the holder again. You must feel it engage.

6. Place the filter holder in its slot in the main unit and push it upwards until it audibly and noticeably clicks into place. Make sure that the inscription **“Vorderseite / Front”** is legible when you insert it (**Fig. E4**).
7. Set up the device and insert the tank into the main unit (see the “Filling the Tank” chapter).

Troubleshooting

If the device does not function properly, check whether you can solve the problem yourself using the following tips. If this is not possible, contact our customer service.

Do not try to repair an electrical device yourself!

Problem	Possible cause	Solution
The device cannot be switched on.	The mains plug is not inserted in the socket correctly.	Correct the fit of the mains plug.
	The socket is defective.	Try another socket.
	There is no mains voltage.	Check the fuse of the mains connection.
The fuse in the fuse box is triggered.	Too many devices connected to the same circuit.	Reduce the number of devices in the circuit.
The device fails to operate.	The device is in standby mode.	Switch on the fan manually.
	The tank (8) is empty.	Fill the tank.
	The timer is deactivated.	Set the timer
	The appliance has fallen over and the safety switch has been activated, automatically putting the appliance in the standby mode.	Place the device on a stable, flat surface.
The display (2) fails to respond.	There is a software problem.	The software has to be reset. Press button ① (4) for approx. 10 seconds. The set time and the timer are also reset.
The device emits a beep every 5 minutes.	The refill reminder is active. / The tank (8) has to be refilled.	Fill the tank and reset the refill reminder.
During use, an unpleasant smell is detected.	The device is being used for the first time.	There are production residues in/on the device. This is normal and will not occur again later. Make sure that the room is well ventilated!

Technical Data

Device name:	DS VieGlobal Thermal Brine Evaporator
Model number:	001-1100-01
Reference article number for the customer service:	08362
Voltage:	230 V
Network frequency	50 Hz
Power consumption:	0.03 A
Power:	2.5 W
Battery:	3 V button cell CR2032
Control unit weight:	1700 g
Fuse:	T3.15A H250 V
Protection level against electric shock:	Type B 
Degree of protection against the penetration of liquid:	IP21
Protection class:	II
Temperature range:	Operation: 5 – 40 °C Storage and transport: -25 – +70 °C
Humidity:	Operation: 30 – 75 % Storage and transport: max 90 %
Air pressure:	Operation: up to 2000 m altitude, 70 – 106 KPa Storage and transport: 70 – 106 KPa
ID of the operating instructions:	Z 08362 M GM V2 0720 dk

Disposal

Dispose of the packaging material in an environmentally friendly manner so that it can be recycled.

This device is governed by the European Directive 2012/19/EU on waste electrical and electronic equipment (WEEE). Do not dispose of this device as normal domestic waste, but rather in an environmentally friendly manner via an officially approved waste disposal company.

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All rights reserved.